DE

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Intensiv ApproxOpener

Produktbeschreibung

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Intensiv ApproxOpener

Product description

- Manual metal strip
- Versions
- Serrated
- Serrated and diamond coated (8µm) on one side
- Serrated and diamond coated (8µm) on both sides
- Height: 4 mm
- Overall length: 20 mm
- Thickness of the serrated strip: 0.08 mm
- Thickness of the serrated and one-side diamond coated strip: approx. 0.09 mm
- Thickness of the serrated and two-side diamond coated strip: approx. 0.10 mm
- Total height of the instrument: 18 mm
- Total length of the instrument: 27.5 mm
- Stainless steel
- Sterilizable and reusable
- Intensiv ApproxOpener is active on one side and passive on the opposite side

Intended use

- Opening of tight interproximal areas in orthodontic treatments (one-side and two-side diamond coated versions).
- Removal of excess material in the interproximal areas.

Practical Tips

- Application by dentist only.
- Intensiv ApproxOpener is positioned incisally/coronally between the teeth in accordance with the interdental space and moved in gingival direction.
- Depending on the initial positioning of Intensiv ApproxOpener in the interdental space, the forward or return movement will be active or passive.
- The removal of excessive restoration material is carried out with the non-diamond coated Intensiv ApproxOpener applied as mentioned above.

Maintenance and Sterilisation

- Intensiv ApproxOpener is provided non-sterile. Before first use it must be sterilized. After the use on the patient the instrument must be cleaned, disinfected and sterilized as well.
- Full cleaning, disinfection, sterilization, checking and maintenance instructions can be requested at info@intensiv.ch

Risk Warnings

- It is recommended not to bend the working part or use it as a lever, applying a force superior than 2N, in order to prevent a break of the instrument.
- Please pay attention to the force applied to the instrument when working in the vicinity of soft tissue; apply a force similar to that used with a manual finishing strip to prevent mucosal lesions.
- Intensiv ApproxOpener is held firmly between fingers and secured with dental floss.

In the unlikely case of an incident caused by the instrument please refer to the manufacturer and competent authorities.

Intensiv SA declines any liability for any use of the instrument other than that specified in these instructions for use.

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