DE

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IPR-DistanceControl Set

Produktbeschreibung

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IPR-DistanceControl Set

Product description

- Intensiv IPR (InterProximalReduction)-DistanceControl is a manual metal instrument
- 8 metallic elements of different thicknesses with ergonomic grip:
 - 0,10 mm
 - 0,15 mm
 - 0,20 mm
 - 0,25 mm
 - 0,30 mm
 - 0,40 mm
 - 0,50 mm
 - 1,00 mm
- Height of metallic element: 4mm
- Length of metallic element: 40mm
- Total length of instrument: 50mm
- Plastic clip as fastening system of the 8 elements in the grip
- Sterilizable and reusable

Intended use

- Support for the control of the teeth distance in the various stages of the IPR technique in orthodontics.

Practical tips

- IPR-DC is inserted in the vestibular/buccal direction.
- If the designated instrument does not enter into the interdental space created, proceed with the instrument of thickness immediately below or through a combination of 2 or more instruments.
- If the designated instrument has too much play in the interdental space created, proceed with the instrument of thickness immediately above or through a combination of 2 or more instruments.
- In order to assemble the Intensiv IPR-DistanceControl Set, insert the plastic rivet inside the holes of the 8 superimposed metal instruments and apply pressure to engage the head in the rivet.
- In order to disassemble the Intensiv IPR-DistanceControl Set, remove the head from the rivet by applying outward pressure.

Maintenance and sterilisation

- Intensiv IPR-DistanceControl is supplied non-sterile. Sterilize the instruments before first use. After each use on patients, the instruments must be disinfected, cleaned and sterilized as well.
- Sterilization should be performed on the instruments without plastic clip.
- Use a single-pulsed or fractionated vacuum autoclave and subvacuum drying.
- Sterilization must be carried out according to validated procedures.
- Full cleaning, disinfection, sterilization, checking and maintenance instructions can be requested at info@intensiv.ch or visualized through the QR-code on the bottom of the page.

Risk Warnings

- During insertion pay attention to the soft tissue to avoid injuries.

In the unlikely case of an incident caused by the instrument please refer to the manufacturer and competent authorities.

Intensiv SA declines any liability for any use of the instrument other than that specified in these instructions for use.

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