DE

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Intensiv Ortho-Strips System

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Intensiv Ortho-Strips System

University of Zurich

Oscillating instruments for easy and rapid opening of tight interdental contact points and for the unilateral and bilateral interproximal reduction of the tooth size and finishing and polishing of interproximal surfaces in Orthodontics.

To be used in combination with the contra-angles:

- Intensiv Swingle, WG-69 LT Ortho, with light
- Intensiv Swingle, WG-69 A Ortho, without light

Alternatively:

- KaVo Kopf 61 LG
- KaVo contra-angle E61C

Product description

- Double-sided diamond-coated oscillating metal strip:
 - Intensiv Ortho-Strips Opener OS08OP-DS: serrated and diamond coated on both sides, 8µm
 - Intensiv Ortho-Strips Extra Coarse OS08XC-DS: perforated and diamond coated on both sides, 80µm
- Intensiv Ortho-Strips Coarse OS60C-DS: diamond coated on both sides, 60µm
- Intensiv Ortho-Strips Medium OS40M-DS: diamond coated on both sides, 40µm
- Intensiv Ortho-Strips Fine OS25F-DS: diamond coated on both sides, 25µm
- Intensiv Ortho-Strips Polisher OS15POL-DS: diamond coated on both sides, 15µm
- One-sided diamond-coated oscillating metal strip (L = coated on left side. R= coated on right side):
 - Intensiv Ortho-Strips Opener, OS08OP-L or OS08OP-R: serrated and diamond coated on one side, 8µm
 - Intensiv Ortho-Strips Extra Coarse, OS08XC-L or OS08XC-R: perforated and diamond coated on one side, 80µm
 - Intensiv Ortho-Strips Coarse, OS60C-L or OS60C-R: diamond coated on one side, 60µm
 - Intensiv Ortho-Strips Medium, OS40M-L or OS40M-R: diamond coated on one side, 40µm
 - Intensiv Ortho-Strips Fine, OS25F-L or OS25F-R: diamond coated on one side, 25µm
 - Intensiv Ortho-Strips Polisher, OS15POL-L or OS15POL-R: diamond coated on one side, 15µm
- Strip height: 3.7 mm
- Length of the diamond-coated part: 13 mm
- Sterilizable and reusable

Intended use

- Unilateral and bilateral initial opening of tight interdental contact points at the beginning of the IPR orthodontics protocol
- Unilateral and bilateral interproximal enamel reduction during the IPR orthodontics protocol
- Unilateral and bilateral finishing and polishing of the interproximal surfaces.

Recommendations for use

- Application by dentist only.
- Apply Intensiv Ortho-Strips Opener at the beginning of the IPR orthodontics protocol for the opening of tight interproximal contact points.
- Subsequently apply Intensiv Ortho-Strips according to the IPR protocol for orthodontic treatments.
- In case of very tight contact points separate the teeth by placing a wooden wedge cervically. This makes stripping more comfortable both for operator and patient.
- Set the micro motor at 40'000 RPM/min max. With Intensiv Swingle this corresponds to 20'000 oscillations per minute approx. (2:1).
- Activate water spray during the entire treatment (minimum 50 ml/min).
- Regulate water flow in case of water excess or shortage.
- Insert the strip in a cervical direction while the contra-angle is moving. This prevents the instruments from jamming.
- Apply a load not greater than 100 grams, up to a maximum bending of the strip of approximately 1 mm.
- Do not stop the strip in the interdental space.
- Remove the strip from the interdental space whilst the contraangle is moving.
- It is advisable to fasten Intensiv Ortho-Strips with dental floss during treatment (loop around arch), for a better control of the instrument position.

Maintenance and Sterilization

- Instruments are delivered non sterile. They must be sterilized prior to first use and disinfected, cleaned, and sterilized after each use.
- Clean the instruments and remove debris after each use, so as to maintain their abrasive properties.
- Use a single-pulsed or fractionated vacuum autoclave and subvacuum drying. Hot air sterilizers are not suitable for diamond instruments.
- Application must be carried out according to validated procedures.
- Full cleaning, disinfection, sterilization, checking and maintenance instructions can be requested at info@intensiv.ch or visualized through the QR-code on the bottom of the page.

Risk Warnings

- Avoid bending and levering: use of excessive force (strong levering effect with the contra-angle) could lead to the breakage of the instrument.
- Avoid jamming or levering actions when oscillating, as this increases the risk of instrument breakage.
- Having passed over the tooth equator line with Intensiv Ortho-Strips Opener, pay attention not to injure the soft tissues.
- Oscillating over 20'000 oscillations per minute could cause breakage of the instrument.
- Avoid applying loads greater than 100 grams as this could cause damage to the instrument and/ or the tooth.
- To ensure traceability of the instruments during their entire application, we recommend keeping the packaging (lot number).

In the unlikely case of an incident caused by the device please refer to the manufacturer and competent authorities.

Intensiv SA declines any liability for any use of the instrument other than that specified in these instructions for use.

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