Intensiv ProxoContour Coarse

DE

Produktbeschreibung

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GB

Product description

- Metal strip, diamond-coated on one sided only with two working parts of different grit sizes: (80µm, blue, 40µm, red)
- One sided, 3 rows of perforation, diamond-coated metal strip
- Perforated anatomical ends for a better gripping
- Height: 2.5mm
- Total length: 80mm
- Thickness: 0.05mm
- Sterilizable and reusable

Indications

- Shaping (80µm) and finishing (40µm) of special excess in proximal area all aesthetic restorations.
- Correction through an efficient and rapid contouring of all existing aesthetic restorations

Practical Tips

- The total length of the strip (80mm) allows an optimal finger support on the adjacent teeth during work.
- To get past the contact area, use the uncoated part of the strip and exert a slight horizontal movement. Do not twist, bend or pull strip too tightly during use.
- After introduction, work with the less abrasive side (40µm, red), in order to facilitate the access of the more abrasive side (80µm, blue). Then begin the finishing procedure using a back-and-forward movement.
- Then apply the 80µm diamond-coated side, with cautious horizontal movements, to model the proximal contours. Next, switch again to the less abrasive side (40µm, red) to complete the procedure with the finishing.
- ProxoContour Coarse must be removed with a vertical movement after repositioning it to the uncoated segment.
- In the event the contact area is very tight, it is useful to start by inserting a wooden wedge to open up a slight separation between the teeth before placing ProxoContour Coarse in the area to be treated.
- After finishing, polish proximal surfaces. We recommend Intensiv ProxoPolish with 15µm and 8µm diamond grit.

Maintenance and sterilisation

- ProxoContour Coarse is supplied non-sterile. Prior to first use on the patient and immediately after each use, the instruments must be disinfected, cleaned, and sterilized.
- Keep ProxoContour Coarse separate from non-stainless steel instruments, such as rubber polishers and abrasive stones.
- If disinfection by immersion is being used, it is important to use only cleaning/disinfection solutions that provide corrosion protection and strictly observe the concentrations recommended by the manufacturer.
- Sterilization must be performed with validated procedures. If possible, use a single-pulsed or fractionated vacuum autoclave and subvacuum drying. Chemiclave sterilizers may also be used.

Risk Warnings

- Tilting or rotating the ProxoContour Coarse metal strip during use can lead to fracture and/or fissure.
- Pulling too forcefully in tight interdental spaces or beginning with the more abrasive (80µm) grit can cause the fissures.
- Bending the ProxoContour Coarse metal strip causes possible breaking points and can lead to detachment of the diamond coating.
- Leaving proximal surfaces roughened with 40µm grain size can lead to undesirable mechanical retention and formation of plaque.

In the unlikely case of an incident caused by the instrument please refer to the manufacturer and competent authorities.

Intensiv SA declines any liability for any use of the instrument other than that specified in these instructions for use.

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